Hufpolster Hoofpac/Mousse de pied/ Materiale di sostegno/Silicone

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How to use our products

Always mix at ratio indicated on packaging. Only this mixture will result in maximised weight-bearing capacity. Alternative: If you use 20% less of the hardener than described, the pad gets softer and more ductile. Using even less of the hardener is not recommended. The ideal temperature of the material is 20-30°C, during cold weather always heat thoroughly to at least 20°C. The warmer the material, the faster the hardening of the pad, during the finishing avoid contact with water, even one drop of water will result in foaming. Storage: do not expose to direct sunlight, protect from freezing. This extends the storage life of the products. Shake well, if the hardener has crystallised this works for up to 30% of crystallisation. More than 30% means the hardener is too old and needs to be replaced. Storage life of the pads in good storage conditions about 15 months. In the car the pads need to be kept in the dark, in a cardboard box or a drawer at all times

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Wolke 7

Can:

Mix Luwex Wolke 7 thoroughly in the ratio of 1:1 until the colors become uniform and apply it to the hoof. At room temperature (20°C) curing time requires 4-5 min.

Cartridge:

Open cartridge and screw on mixing tip. Trim back mixing tip to enlarge output hole to approximately 8 mm. Do not use first 2-3 ml (first splash), as blend will not be optimal. Fill hoof with Cloud 9. Remove mixing tip after use and close cartridge with original closing plug and cap nut. Best processing temperature is 30°-40°C. To speed up curing time, place cartridge in warm water for 15 min. or preheat with heat gun.

120 S

Best processing temperature is 25°C; curing time will be longer in cold working condition and shorter in warm condition. Put the appropriate amount for one hoof in a cup (you can measure amount directly in the hoof). Add activator and mix well and fast for 20-30 sec. Work fast - Hoofpac bonds within 3 min. Flatten Hoofpac with spatula (use only as much as necessary). Nail on prepared Shoe with Pad, using the toenails only. Let horse stand on foot briefly. Flatten excess material in the heel area and finish nailing on. Mix Hoofpac only for one foot at a time!
Work cleanly, avoid contact with hands and tools. Use dishwater (0,1 l water and 2 tablespoon full of dishwashing agent) or spit for flattening.

M 15 XXL

Processing temperature 20°-35°C, preheat M 15 when using in cold weather. Squirt 18-24 ml activator into bottle (preferably with a syringe). Shake strongly or mix for at least 10 sec. Always shake M 15 and activator well and immediately incorporate mixture into the hoof. When using M 15 with a pad, cut a 14 mm hole into the pad.

M 9

Bottle:

1. Disinfect sole with LUWEX Hoofspray or similar. 2. Nail on shoe with meshpad and seal heel area with adhesive tape. 3. Fill bottle up to marker with activator and shake approx. 10 -15 sec. fast and thoroughly. Cut off tip diagonally and pour in M 9 immediately. For this purpose slip tip through mesh of pad and squeeze bottle strongly or remove tip completely. Work quickly: Adding activator, mixing and pouring in should be done within 30 sec. M 9 is rigid after 30 more seconds and after additional 30 seconds the horse can stand on its foot again. Best processing temperature is 25°C.

Cartridge:

Open Cartridge and screw on mixing tip. Do not use first 2-3 ml (first splash), as blend will not be optimal. Fill up hoof with M 9. Remove mixing tip after use and close cartridge with original closing plug and cap nut. Set time (30°C) approx. 30 sec., colder working condition prolongs set time. Best processing temperature is 20°-30°C.

Hoof Gel MB 12

Processing temperature 20°-35°C; always preheat MB 12 when using in cold weather. Squirt 16 ml activator into MB 12 special bottle (preferably with a syringe) and shake strongly or mix for 10 sec. Immediately pour mixture into hoof. When using a pad, cut a 14 mm hole into the pad, remove screw cap from bottle and pour MB 12 directly into the hole. Set time (20°C) approx. 60 sec., colder working condition prolongs set time a little.

M 20

Mixing ratio 10:3 (100ml M 20: 30 ml activator)

Fill up bottle (60ml) with 18 ml activator, shake bottle fast and strong and immediately pour M 20 into the prepared foot. Rigid after approx. 60 sec., after additional 60 sec. the horse may stand on its foot again. Mask molded bar from M 20 with adhesive tape or besprinkle it with sawdust or chalk dust to prevent M 20 from adhering to the ground.

Best processing temperature is 20°-30°C.

Ideally the M 20-Soft Bar encloses the mesh pad 1 cm towards the center of the foot. This extends the durability of the soft bar.

PREMIUM

Simply mix with your hands, set time at room temperature is approx. 3 min.

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